

Relaxa-Back



The Relaxa-Back will give support when getting onto the belt and lowering into the bath.



When the Relaxa-Back is moved to the back of the bath, it can be used as a head rest.



RELAX WITH A **Relaxa**TM BATHLIFT

- Easy to fit
- Battery powered NOT mains Electricity
- Audible alarm when battery is low
- Remote control as standard
- Thermometer for water temperature
- Does not obstruct the bath
- Provides independence for those that find the bath difficult to enter or leave

FOR FURTHER INFORMATION PLEASE CONTACT

CR
MANUFACTURING

C.R. Manufacturing Ltd

Frederick Street, Waddesdon, Aylesbury, Bucks HP18 0LU


Tel: (01296) 655001 Fax: (01296) 658441


crmanufacturing@btconnect.com www.crmanufacturing.co.uk


The company policy is one of continuous improvement and reserves the right to change the design and specification without notice.



Quality
Made in the UK


HOW DOES IT WORK

Simply push in the re-chargeable battery. 

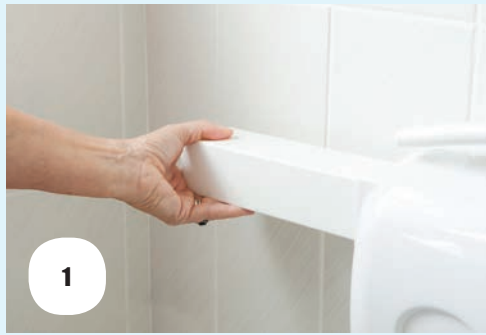



Sit on the belt and by touching a button, be gently lowered into the bath. 

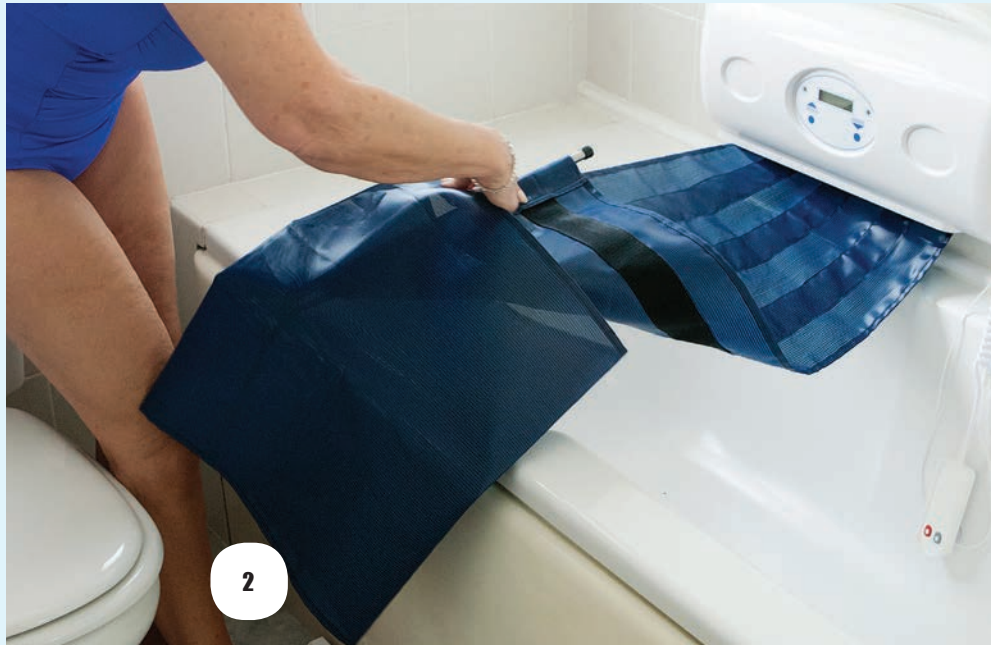



Feed out the belt and slip on belt sleeve. 

Relax & enjoy the luxury of a good soak. 

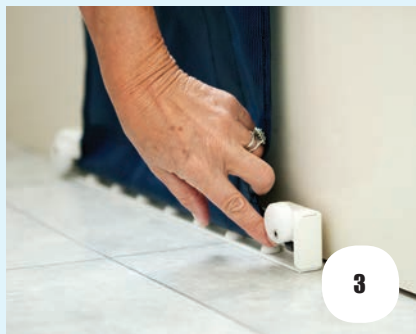


Locate the belt on the floor mounting bar and tension. 

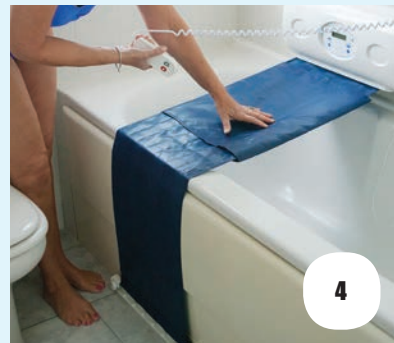

Touch a button and be gently raised out of the bath. 



Audible alarm + Graphics when battery is low. 



In addition there is a remote hand control and a temperature probe to register the water temperature and avoid scalding. 